

Syringe Covers



- Avoid a lengthy cleaning process of dose calibrator and dipper
- Designed to assist in compliance to USP Chapter 825 guidelines
- Fits up to one 10cc Syringe

SC-100 Syringe Covers (100 per package)

Sterile 70% USP Isopropyl Alcohol



- 16 oz spray bottle
- Ships continental US only

S70-16 Sterile 70% USP Isopropyl Alcohol, Spray Bottle

Syringe Cover Container



- Easy access container to keep your syringe covers from being misplaced
- Fits up to one package of SC-100 Syringe Covers (100 per package)

SC-101 Syringe Cover Container

Humidity and Temperature Indicator



- Precalibrated to NIST-Traceable standards
- Displays Humidity levels, Temperature, and Time
- Reads in Fahrenheit and Celsius

HTI Humidity and Temperature Indicator

USP 825 requires technologists to clean their dose calibrator well insert and dipper when used for measuring radiolabeled blood products or residual activity post administration.

Our new disposable syringe cover with integrated seal (Model SC-100) will keep you compliant! Simply place the syringe inside a disposable syringe cover, seal it, spray the exterior with sterile 70% USP isopropyl alcohol (Model S70-16), and proceed with the assay. It's that easy, the cleaned sealed cover keeps the assaying of your syringes USP 825 compliant.

825 Starter Pack

Be prepared for USP Chapter 825 with all of the above products!

- Two packages model SC-100 Syringe Covers (100 per package)
- Model SC-101 Syringe Cover Container
- Model S70-16 Sterile 70% Isopropyl Alcohol Spray, 16 oz



SC-102 825 Product Starter Pack